
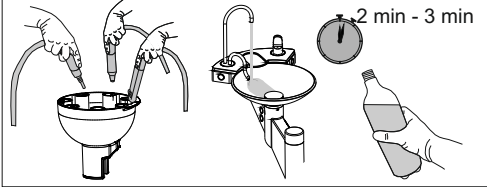
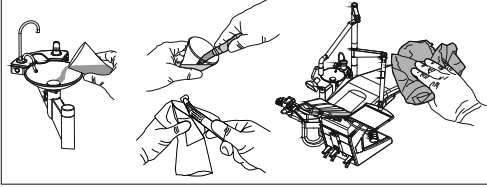
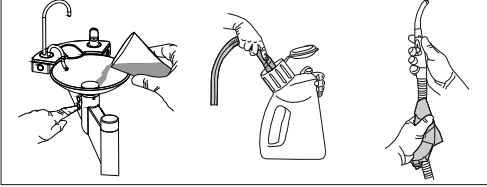
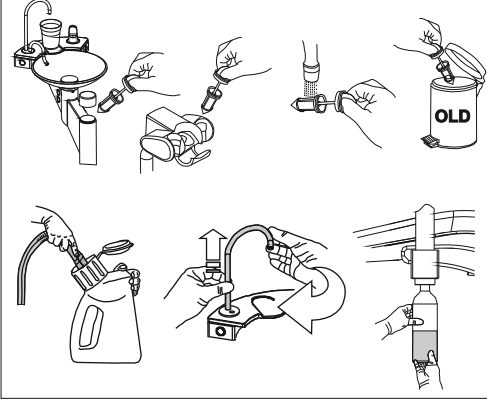


When?	What?	How?
In the morning 	<ul style="list-style-type: none"> ■ Instrument hoses and water-bearing system ■ glass filler, spittoon ■ Suction hoses (inside) ■ Bottlesystem (if available) 	<ul style="list-style-type: none"> ▶ Rinse water-carrying instruments in the CleanHub¹⁾ (without instruments) ▶ Activate glass filler and spittoon (via automatic rinsing program CleanHub) ▶ Suction cleaning with water ▶ Fill with fresh water (if necessary by adding liquid concentrate for disinfection, e.g. Alpron)
After each treatment 	<ul style="list-style-type: none"> ■ Suction system and spittoon ■ Cleaning and disinfection treatment surfaces (<i>without upholstery</i>) ■ Disinfection of the upholstery ■ Disinfection of instruments 	<ul style="list-style-type: none"> ▶ Aspirate a glass of water on the drain system and the suction system ▶ Manual cleaning and disinfectant (e.g. FD322/FD350/FD366) ▶ Manual disinfectant (e.g. FD312, FD366) ▶ Manual disinfectant (e.g. FD333, FD350)
In The Evening 	<ul style="list-style-type: none"> ■ Cleaning and disinfection of suction hoses (inside), aspirations system and spittoon ■ Suction hoses outside and Instrument hoses outside 	<ul style="list-style-type: none"> ▶ Non-foaming material acceptable disinfectant (e.g. Orotol Plus, Orotol ultra) ▶ Manual cleaning and disinfectant (e.g. FD322/FD350/FD366)
Weekly 	<ul style="list-style-type: none"> ■ Filter inserts in spittoon and in the hose holder system ■ Suctionsystem/Suction hoses (inside) ■ Spittoon attached components ■ Bottlesystem (if available) 	<ul style="list-style-type: none"> ▶ Remove the filter cover, extract the yellow filter rinse it under water or change and replace it. Do not rinse the filter in the spittoon ▶ Non-foaming material acceptable disinfectant (e.g. MD555 cleaner) ▶ Remove components (glasfiller and flushing pipe) and clean them, lubricate O-Rings (320.3981.01) ▶ Glasfiller and flushing pipe put for sterilise in Bilpron (follow manufacturer's instructions) ▶ Rinse all extraction points with liquid concentrate for disinfection (no mounted instruments) (e.g. Bilpron)
	<ul style="list-style-type: none"> ■ Cleaning the upholstery ■ Cleaning the bottel (inside) 	<ul style="list-style-type: none"> ▶ Manual cleaning (e.g. FD360) ▶ Manual cleaning (e.g. BC-San 100)

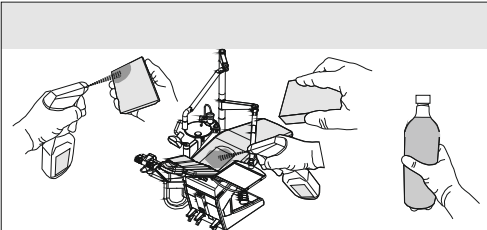

Please use only Mikrona recommended disinfectant!

¹⁾Recommended by RKI: 2 Min. Rinse all extraction points without attached instruments.

This care and cleaning instruction includes only the essential operating functions. It does not replace the official Instruction manual.