# MIGMA 200

## INSTALLATION

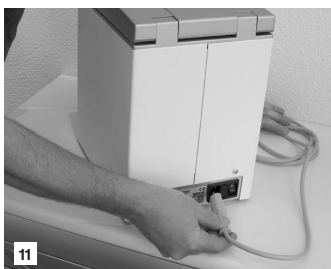
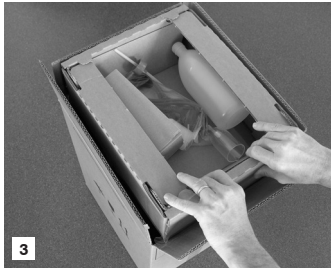
D

## INSTALLATION

E

## INSTALLATION

F

### Anleitung

D

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### Instructions

E

- 1 Open the box from the top.
- 2 Container with accessories.
- 3 Remove the container with accessories.
- 4 Remove the side padding (2 pieces).
- 5 Lift the device out of the cardboard box.
- 6 Place the device on a working surface.
- 7 Tilt the device to the side and remove the carriage safety screws.
- 8 Only tilt the device slightly when removing the last safety screw.
- 9 Remove the information tape.
- 10 Open the cover and remove the cardboard tube.
- 11 Connect the device to the power supply system (100 – 230VAC).
- 12 The device is ready for operation.

### Retain original packaging (Art.-No. 430.6027.02)!

The mixing device MIGMA 200 is to be shipped in original packaging only. Retain the cardboard box incl. containers, padding and safety screws.

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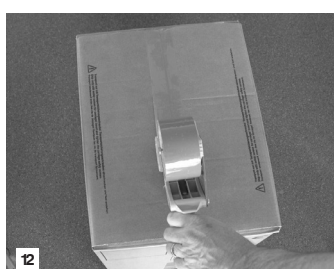
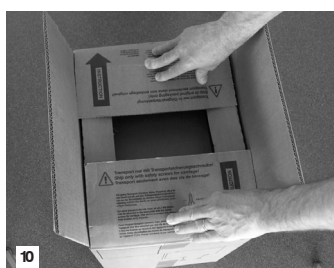
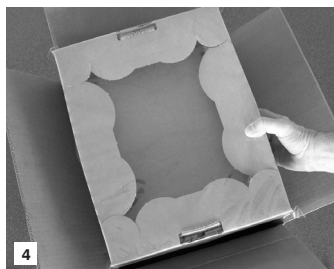
### Instructions

E

- 1 Open the cover and insert the cardboard tube.
  - 2 Turn the device upside down.  
Caution: Hold cover closed during turning!
  - 3 Insert and tighten the carriage safety screws.
  - 4 Piece of padding into bottom of the box.
  - 5 Second Piece of padding into the box.
  - 6 Place the device in the box center.
  - 7 Insert padding on the sides (2 pieces).
  - 8 Insert the upper support.
  - 9 Press down first side of packaging.
  - 10 Press down second side of packaging.
  - 11 Close the cardboard box.
  - 12 Close the box tightly. Ship the device in original packaging only.
- ⚠ **Ship in original packaging only (Art.-No. 430.6027.02)!**  
The mixing device MIGMA 200 is to be shipped in original packaging only. For each transport, the safety screws and the cardboard tube are to be inserted. We assume no liability for damages resulting from non-observance of this instruction.

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